

Klarity SRS Adaptor

INSTRUCTIONS FOR USE

GENERAL INFORMATION

The Klarity SRS Adaptor converts an existing S-type Baseplate into an SRS-compatible overlay. It is used for positioning and re-positioning patients during radiation therapy treatment.

PRODUCT DESCRIPTION

The Klarity SRS Adaptor clicks into your existing S-type baseplate just like a mask. With the SRS Adaptor in place, both bottom and top SRS masks can be easily and securely attached. Mask height can be adjusted between 0-4mm at each pin location using the Simple Shim System™ included with Klarity SRS Masks.


The SRS Adaptor is compatible with SRS Masks from Klarity. Please learn more about Klarity SRS Masks online at www.klaritymedical.com/srs.

STORAGE AND CLEANING

Store in a clean, dry place away from direct sunlight. Wipe thoroughly with a neutral antiseptic cleaner. Allow to dry thoroughly before next use.

INSTRUCTIONS FOR USE

1. Attach the S-type baseplate to the treatment table or CT Couch using an indexing bar.
2. Position the SRS Adaptor on the baseplate and click the pins in to secure the Adaptor.
3. Once the Adaptor is in place, it is ready for the Klarity SRS Mask. Please refer to the Klarity SRS Mask Set IFU for full instructions.

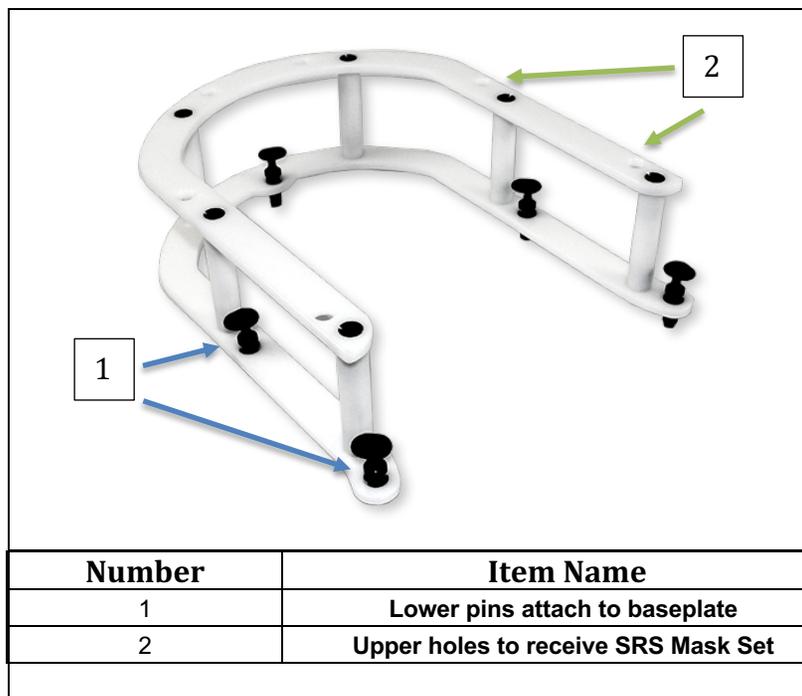

To be used by a qualified medical professional
IFU-B-001



EC REP

Lotus NL B.V
Koningin Julianaplein 10, le Verd, 2595AA
The Hague, Netherlands
T: +31645171879 (English)
T: +31626669008 (Dutch)

Klarity Medical Products, LLC
600 Industrial Parkway
Heath, OH 43056
T: 740.788.8107 F: 740-788-8109
info@klaritymedical.com
www.klaritymedical.com