

# Klarity PelvicFix™ Baseplate INSTRUCTIONS FOR USE

## **GENERAL INFORMATION**

The Klarity PelvicFix<sup>™</sup> Baseplate is used for positioning and repositioning patients during radiation therapy treatment.

## PRODUCT DESCRIPTION


The Klarity PelvicFix™ Baseplate is used for immobilizing the pelvis during radiation therapy treatments. The baseplate fixes to the treatment couch with a standard indexing bar. It is compatible with Hip & Pelvic Masks from Klarity. Please see our full mask selection at <a href="https://www.klaritymedical.com/masks">www.klaritymedical.com/masks</a>.

#### STORAGE AND CLEANING

Store in a clean, dry place away from direct sunlight. Wipe thoroughly with a neutral antiseptic cleaner. Allow to dry thoroughly before next use.

#### INSTRUCTIONS FOR USE

- 1. Attach the baseplate to the treatment table or CT Couch using an indexing bar.
- 2. Position the patient on the baseplate
- 3. Pull the sliders out so that you can insert a mask onto the baseplate.
- Prepare your thermoplastic mask per the manufacturers' instructions
- 5. Place the thermoplastic mask directly over the patient and connect the frame to the baseplate.
- 6. Bring the sliders in to fix the frame to the baseplate.
- 7. If using the optional Groin Attachment, attach the white pins on top of the pelvic mask holes, and secure in place with the slider.
- 8. Once the mask cools, release the mask by pulling the sliders out and lifting the mask.

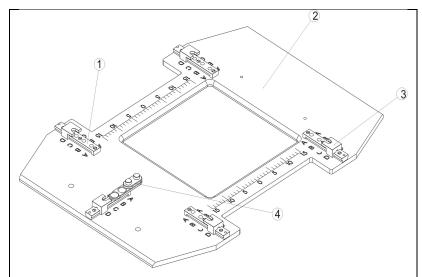

Number	Item Name
1	Silder Bar
2	Baseplate
3	Slider
4	Groin Locating Piece

To be used by a qualified medical professional IFU-B-007















