

Klarity Duo Align System

INSTRUCTIONS FOR USE

GENERAL INFORMATION

The Klarity Duo Align System is used for positioning and re-positioning patients during radiation therapy treatment.

PRODUCT DESCRIPTION


The Klarity Duo Align System is an AIO (all in one) baseplate used for immobilizing and supporting the patient during radiation therapy treatments. The baseplate fixes to the treatment couch with two indexing bars (upper and lower). It is compatible with O-type and U-frame masks for Head and Shoulder immobilization, and it is also compatible with thoracic and pelvic masks for full body positioning. It is compatible with 3 optional accessories: Groin Lock, Overhead Arm Positioner, and Leg Cushions.

STORAGE AND CLEANING

Store in a clean, dry place away from direct sunlight. Wipe thoroughly with a neutral antiseptic cleaner. Allow to dry thoroughly before next use.

INSTRUCTIONS FOR USE

1. Attach the baseplate to the treatment table or CT Couch using two indexing bars.
2. Place the desired headrest onto the baseplate.
3. Position the patient onto the baseplate.
4. Place the mask onto the patient and draw directly down onto the locating pins.
5. If using U-frame masks, unscrew the clamps so that they can rotate over the mask frame, then tighten to secure. If using O-type masks, insert into the slot and place the block alongside to secure.
6. Allow mask to cool completely before removing.

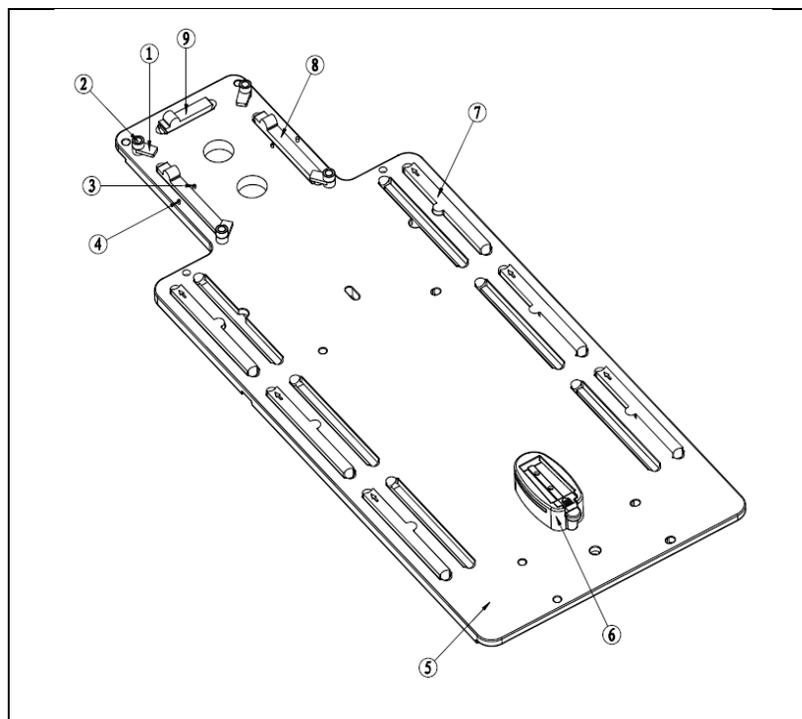

| Number | Item Name |
|--------|----------------------------------|
| 1 | Rotating clamp for U-frame Masks |
| 2 | Clamp Screw |
| 3 | Silverman Pins |
| 4 | Locating Pin for U-frame Masks |
| 5 | Baseplate |
| 6 | Groin Lock (optional) |
| 7 | Indexing Slot (232mm length) |
| 8 | Indexing Slot (178mm length) |
| 9 | Indexing Slot (98mm length) |

To be used by a qualified medical professional
IFU-B-010



Keep in a cool and dry environment



Store below 35°C (95°F)



Lotus NL B.V
Koningin Julianaplein 10, le Verd, 2595AA
The Hague, Netherlands
T: +31645171879 (English)
T: +31626669008 (Dutch)

Klarity Medical Products, LLC
600 Industrial Parkway
Heath, OH 43056
T: 740.788.8107 F: 740-788-8109
info@klaritymedical.com
www.klaritymedical.com