



## Instructions for Use

Follow current department protocols regarding intradermal procedures.


### VACCINATION TECHNIQUE

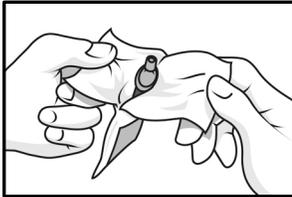

**1.** Verify that Steritatt package is completely sealed and unopened. Remove Steritatt device from packaging. Hold down on cap and shake firmly.

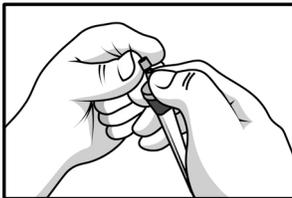


**2.** Remove and discard plastic cap.

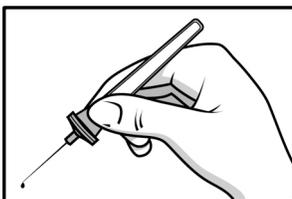


**3.** Attach needle to Steritatt device. Best results are achieved with Short Stem 21 Gauge needle.

Hold SteriTatt at ink bulb between thumb and index finger. Over trash receptacle, squeeze to produce ink drops from needle tip.

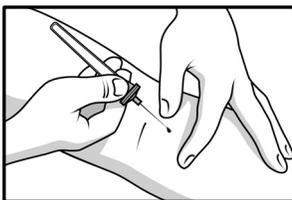

**4.** With free hand, hold patient's skin taut over tattoo point.


Gently insert needle point (lumen) under skin surface. Carefully squeeze ink bulb, visualize ink collection at skin surface, and remove needle.\*

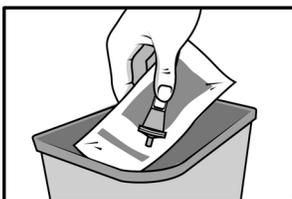


**5.** Discard needle in proper container, SteriTatt in normal trash.needle tip.

\*For a slightly larger tattoo, gently twist needle after introducing ink.

### DROPPING TECHNIQUE

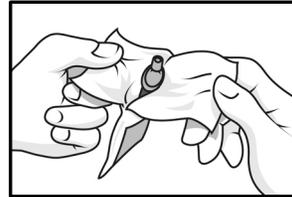

**1.** Verify that Steritatt package is completely sealed and unopened. Remove Steritatt device from packaging. Hold down on cap and shake firmly.

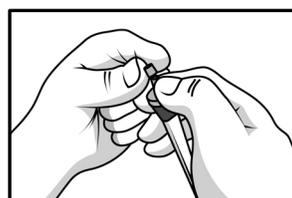


**2.** Remove and discard plastic cap.



**3.** Hold SteriTatt at ink bulb between thumb and index finger. Over trash receptacle, squeeze to produce ink drops from tip.



**4.** Gently insert desired needle point (lumen) under skin surface. Carefully squeeze ink bulb, dripping ink onto needle tip. Twist needle and remove.

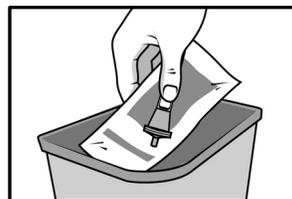

**5.** Discard needle in proper container, SteriTatt in normal trash.needle tip.